Edwards Lifesciences

COMM-DOCU 850/003-C

DIRECTIONS FOR USE
MODE D'EMPLOI
GEBRAUCHSANWEISUNG
INSTRUCCIONES DE USO
MODO DE USAR
MODO D'IMPIEGO
GEBRUIKSAANWIJZING
BRUKSANVISNING
BRUKSANVISNING
BRUGSANVISNING
ΤΡΟΠΟΣ ΧΡΗΣΗΣ
KASUTUSJUHEND
LIETOŠANAS INSTRUKCIJA
NAUDOJIMO INSTRUKCIJA
INSTRUCȚIUNI DE UTILIZARE
NÁVOD NA POUŽITIE
KULLANIM TALĪMATLARI
HASZNÁLATI ÚTMUTATÓ
INSTRUKCJA OBSŁUGI
KÄYTTÖOHJE
ИНСТРУКЦИИ ЗА УПОТРЕБА
NÁVOD K POUŽITÍ
РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ
UPUTSTVO ZA UPOTREBU

as per EEC directive 93/42

DIRECTIONS FOR USE

IDENTIFICATION - APPLICATIONS

Your "pressure cable" plays a critical part in the transmission of the "invasive" blood pressure electric signal.

It links the single use pressure sensor (transducer) to the appropriate monitor.

INTEGRAL PROCESS proposes a full line of high-quality cables complying with the recommendations currently in force.

IMPORTANT

Refer to the installation instructions and directions for use before using the "pressure cable".

Use of the pressure cable by unqualified personnel is prohibited.

For subsequent orders, please use the code number(s) marked on your INTEGRAL PROCESS "pressure cable" or its packaging, for example : 898083001.

Also refer to the directions for use of the monitor to which the "pressure cable" is connected.

MECHANICAL INTEGRITY

The insulation on an INTEGRAL PROCESS cable and on its conductors is made of polyurethane, giving it excellent tensile and bending strength; most of our connectors are molded and have flexible sleeves to minimize the risk of breakage of the conductors.

SAFETY

The INTEGRAL PROCESS connection system does not include any accessible metallic parts.

The contact pins are gold-plated for better contact and are recessed to protect patients from any contact between these pins and live external metallic parts.

CLEANING / DISINFECTION / STERILIZATION

CLEANING

As often as necessary, clean the insulating parts of the "pressure cable" with warm soapy water, then rinse thoroughly.

DISINFECTION

Intensive use of the "pressure cable" requires disinfecting at least once a week in accordance with the following recommendations, taken from the study of the ANIOS laboratory (reference no. 64 16.94/0387).

1. Fill a soaking tank with an 0.5% solution of HEXANIOS G+R solution.
 2. Partially immerse the cable (protect the ends to avoid any electrical trouble with the connectors).
 3. Allow a contact time of 15 minutes.
 4. Rinse the cable.
 5. Dry the cable with absorbent tissue.
 6. During the immersion, rub the ends of the cable with a LINGET'ANIOS.
- N.B.:** Change the solution every 48 hours.

STERILIZATION

A sterilization procedure may have to be performed if the "pressure cables" are used in a sterile room.

INTEGRAL PROCESS recommends sterilizing the cables by the following process after they have been disinfected as described above.

- sterilization time: 22 hours, including 20 hours' exposure to the gas sterilizing agent: carbon dioxide + ethylene oxide (80/20)
- initial vacuum: -70 kPa
- relative humidity: > 60%
- temperature: 50°C
- concentration of ethylene oxide: 530 g/m³
- pressure: 120 kPa
- final vacuum: -70 kPa
- rinsing.

Investigation protocol 003: sterilization test/pressure cables - AQ/VD/IP/SC/PI - Revision 0 of 3/06/98

Never sterilize the "pressure cables" with steam or boiling water.

INSTALLATION INSTRUCTIONS

Check the reference of the "pressure cable", marked on the packaging or on the cable itself, to make sure that it is compatible with the monitor.

To disconnect the cable from the monitor, pull on the connector, not the cable itself.

For the connections of the cable to the monitor and to the pressure sensor, refer to their respective directions for use.

When the "pressure cable" is not in use, it is important to make sure no liquid gets into the connectors. Moisture in the connector can cause the cable to malfunction or alter the pressure reading.

WARRANTY

Unused cables still in their original packaging are guaranteed by INTEGRAL PROCESS for one year counting from the date of manufacture, unless damaged by mishandling.

INTEGRAL PRESSURE cannot guarantee the cable against damage during use.

INTEGRAL PRESSURE guarantees the compliance of the cable with the specifications of the safety and performance standards applicable to it and currently in force.

USE - MAINTENANCE

A reduced operating test is necessary before each use.

The service life of the cable depends on the conditions of use. It is absolutely necessary to follow strictly the checking and verification instructions recommended by the respective manufacturers or the regulations for

equipment used at the beginning of each day of work and before each use. Using an alternative measuring method, make sure the vital parameters of the patient are constant. If in doubt, replace the cable.

Pending use, pressure cables must be kept in their original packaging and stored under the following conditions:

- Ambient temperature between -40 and +70°C
- Relative humidity between 10 and 100%, including condensation
- Atmospheric pressure between 500 and 1080 hPa

The conditions of use are as follows:

- Ambient temperature between +10 and +40°C
- Relative humidity between 30 and 75%
- Atmospheric pressure between 700 and 1080 hPa
- Temperature of water entering a device so cooled, not more than 25°C

Depending on the number of disinfections/sterilizations:

- Visually check the integrity of the cable's insulating sheath.
- Check the electrical continuity of the conductors of the cable.
- Check that the marking is legible.
- Check the mechanical integrity of the connectors.

If the appearance of one or more parts of the cable is doubtful, the cable must be replaced. INTEGRAL PROCESS cannot be held liable for any failure to observe these rules.

INTEGRAL PROCESS will not be held liable for trouble resulting from failure to follow the rules for installing and using the "pressure cable" described in these directions for use.

INTEGRAL PROCESS
12 rue des Cayennes - Z.A. des Boutries
78703 CONFLANS SAINTE HONORINE

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